SentiCath Patient Electrode Kit 5.0

Table of Contents

Sommaire / Inhalt / Sommario / Índice / Índice / Tartalomjegyzék / Spis treści / Obsah /
Съдържание / Sadržaj / Satura rādītājs / 目錄

SentiCath Patient Electrode Kit 5.0

English	3
Français	4
Deutsch	5
Italiano	6
Español	7
Português	8
Magyar	9
Polski	10
Slovenčina	11
Български	12
Hrvatski	13
Latviski	14
繁體中文	15
Symbol Glossary	16

Glossaire des symboles / Symbolglossar / Glossario dei simboli / Glosario de símbolos / Glossário de símbolos /
Szimbólum glosszárú / Glosariusz symboli / Slovník symbolov / Речник на символите / Glosar simbola /
Simbolu vārdnīca / 符號說明

SentiCath Patient Electrode Kit

The SentiCath Patient Electrode Kit 5.0 is designed to be used with the AcQMap® High Resolution Imaging and Mapping Systems 900000 and 900100 to provide cardiac signals, AcQMap® localization and system reference.

The SentiCath Patient Electrode Kit 5.0, Model 459639, includes the following components:

- Monitoring Electrodes (15)
 - 3M Red Dot™ 2670-5 (3)
- Localization Electrodes (6) and System Reference (1)
 - Acutus Medical®

WARNINGS AND PRECAUTIONS

Patient Electrodes - To avoid patient injury, use care in applying and removing Patient Electrodes (Monitoring and Localization).

- To avoid patient injury, the System Reference Electrode ↓ must be the first Patient Electrode connected to the AcQMap® Console at the beginning of the study and the last Patient Electrode to be disconnected at the end of the study. Remove the System Reference Electrode ↓ from the patient's skin prior to disconnecting the electrode lead-wire from the AcQMap® Console.
- Ensure that all Patient Electrodes and connections are not in contact with each other or any other surface electrodes from other equipment (e.g., ablation return electrodes, defibrillation electrodes), electrical ground or metallic objects.
- Do not warm the Monitoring Electrodes or the Localization Electrodes prior to application to the patient.
- Do not use any Patient Electrodes if the packaging seal is not intact, the conductive adhesive is dry, or the "Use By" date has passed.
- Before applying Patient Electrodes, ensure the application site is hair-free, clean and dry.
- Re-use and repositioning of disposable electrodes may result in degradation of performance of the AcQMap® High Resolution Imaging and Mapping System.
- Do not place electrodes on skin folds, dry or damaged skin.
- Do not modify electrodes prior to use.
- MRI compatibility for the electrodes contained in the SentiCath Patient Electrode Kit 5.0 has not been tested by Acutus Medical.

PATIENT ELECTRODE PLACEMENT

Refer to Figure 1 and align the electrodes per the gridlines for the correct placement of the Patient Electrodes. When placing the electrodes, ensure that the cables are directed to the side of the table where the AcQMap® Console is located. Start with the patient sitting upright on the fluoroscopy table.

- Place the System Reference Electrode ↓ on the lower right back. (Figure 1) Connect the System Reference Electrode ↓ to the front panel of the AcQMap® Console.
- Place Localization Electrode 5 (yellow) on the patient's back in a horizontal position with the superior border of the patch at the level of T3. (Figure 1)
- Place Localization Electrode 3 (red) in a horizontal position across the lower back. (Figure 1) This electrode will be parallel to Electrode 6. (see Step f)
- Ensure that both Localization Electrodes are flat and have adequate adhesion to the patient's skin. Help the patient lay down and route the connector cables to the same side as the AcQMap® Console.
- Place Localization Electrode 4 (green) in a horizontal position with the superior edge at the level of the sternal notch. (Figure 1)
- Place Localization Electrode 6 (orange) in a horizontal position across the abdomen midway between the xiphoid process and umbilicus. (Figure 1)
- Place Localization Electrode 2 (gray) in a vertical position across the right ribs. (Figure 1) This electrode should be centered on the heart.
- Place Localization Electrode 1 (black) in a vertical position across the left ribs. (Figure 1) This electrode should be centered on the heart.
- Connect all Localization Electrodes 1 through 6 to their color-coded receptacles on the front panel of the AcQMap® Console.
- Place the ten Monitoring Electrodes as shown in Figure 1.
- Connect the Monitoring Electrodes to the AcQMap® Console using the AcQMap® EGM Input Cable.

NOTE: If at any time during the study, the SentiCath 3D Imaging and Mapping Catheter appears flat (i.e., 2 dimensional), the most likely cause is a poorly attached or poorly located Localization Electrodes. The Localization Electrodes should be inspected as soon as possible and replaced if necessary. After replacing any Localization Electrode, a new anatomy should be generated.

NOTE: AcQMap® System 900000 does not require a right leg electrode.

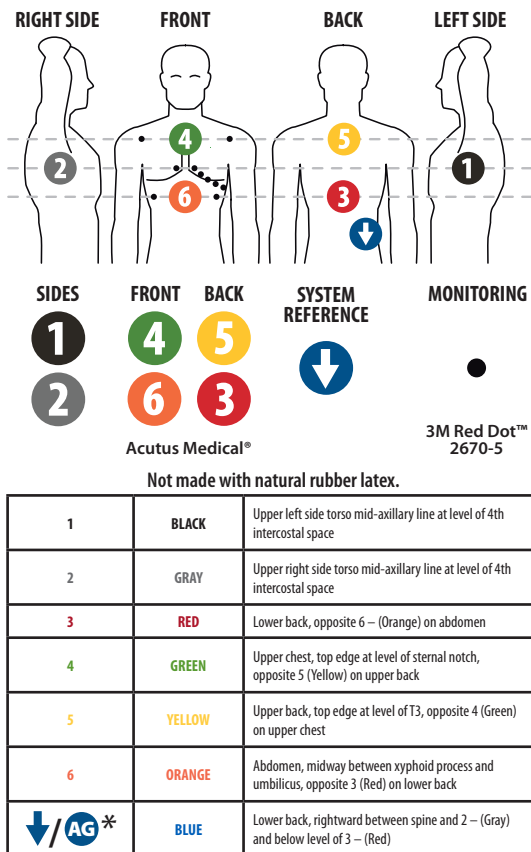
Figure 1. Placement of Localization and Monitoring Electrodes.

*The Blue Arrow symbol replaces the AG symbol previously used in system 900000.

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Acutus Medical, Inc.
MDSS GmbH